
INSTRUCTIONS FOR USE

HOLDER FOR BACKUP PUMP DRIVE

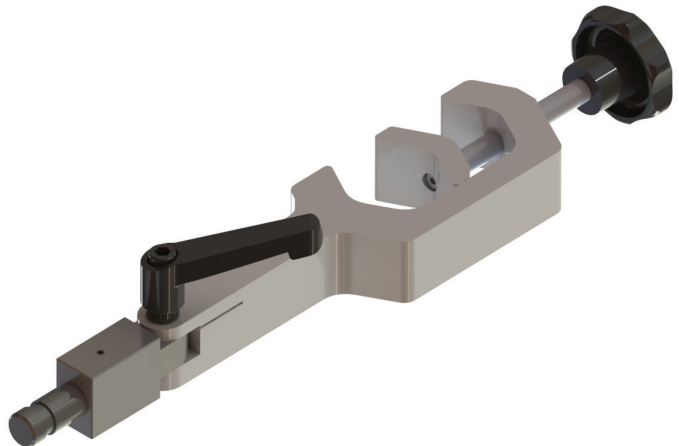CE

MANUFACTURER

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IFU Version:
CE_003_2021_07

Item number IFU:
38350406

Should questions arise or problems/malfunctions occur, note the affected serial number and contact Xenios AG.

Technical Service group +49 7131 2706-234
technicalsupport@xenios-ag.com


Customer Service group +49 7131 2706-100
customerservice@xenios-ag.com

If the Holder for Backup Pump Drive is used incorrectly or for purposes other than those defined in these Instructions for Use, the sole responsibility is with the user.


These Instructions for Use are provided for all qualified medical staff required to work with the Holder for Backup Pump Drive. For safe and proper use, follow these instructions. Keep them for future reference.

A digital version of this document can be found at this address:
<https://www.xenios-ag.com/ifu/>

1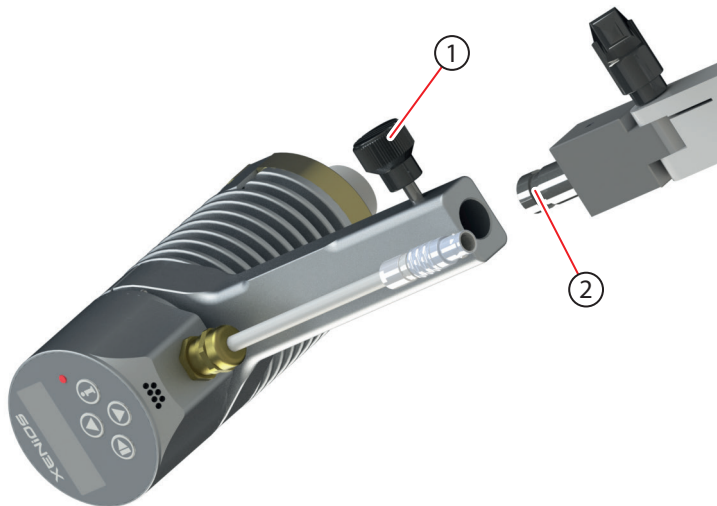

2

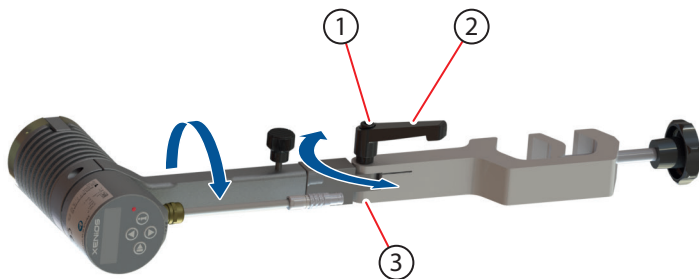SYMBOLS GLOSSARY

MD	<p>Medical device Medizinprodukt جهاز طبي Zdravotnický prostředek Ιατροτεχνολογικό προϊόν Medicinsk udstyr Producto sanitario Lääkinnällinen laite Dispositif médical Orvostechnikai eszköz Dispositivo medico 의료기기 Medisch hulpmiddel Wyrób medyczny Dispositivo médico Dispozitiv medical Медицинское устройство Medicinteknisk produkt Tibbi cihaz 医疗器械 醫療器材</p>
UDI	<p>Unique device identifier Eindeutige Gerätekennung معرف الجهاز الفريد Jedinečný identifikátor prostředku Unik enhedsidentifikation Μοναδικό αναγνωριστικό συσκευής Identificador único del dispositivo Yksilöllinen laitetunniste Identificateur de dispositif unique Egyedi eszközazonosító Identificazione unica del dispositivo 고유 기기 식별자 Unieke apparaatidentificatie Identyfikator UDI (ang. Unique Device Identifier) Identificador de dispositivo único Identificator unic al dispozitivului Уникальный идентификатор устройства Unik produktidentifiziere Benzersiz cihaz kimliği 唯一设备标识符 唯一裝置識別碼</p>
REF	<p>Catalog number Artikelnummer رقم الكتالوج Katalogové číslo Varenummer Αριθμός καταλόγου Referencia Tuotenumero Référence produit Katalógusszám Numero di catalogo 주문번호 Catalogusnummer Numer katalogowy Referência de catálogo Număr de catalog Каталогный номер Artikelnummer Katalog numarası 目录编号 目錄編號</p>
SN	<p>Serial number Seriennummer الرقم التسلسلي Sériové číslo Serienummer Σειριακός αριθμός Número de serie Sarjanumero Numéro de série Sorozatszám Numero di serie 일련 번호 Serienummer Numer seryjny Número de série Număr de serie Серийный номер Serienummer Seri numarası 序号 序號</p>

	<p>Manufacturer Hersteller جهة الصنع Výrobce Producent Κατασκευαστής Fabricante Valmistaja Fabricant Gyártó Produttore 제조자 Fabrikant Producent Fabricante Producător Производитель Tillverkare Üretici 制造商 製造商</p>
	<p>Date of manufacture Herstellungsdatum تاريخ التصنيع Datum výroby Fabrikationsdato Ημερομηνία κατασκευής Fecha de fabricación Valmistuspäivä Date de fabrication Gyártás dátuma Data di produzione 제조일 Productiedatum Data produkcyj Data de producção Data fabricației Дата изготовления Tillverkningsdatum Üretim tarihi 生产日期 生産日期</p>
	<p>Follow instructions for use Gebrauchsanweisung befolgen اتبع تعليمات الاستخدام Postupujte podle návodu k použití Følg brugsanvisningen Ακολουθήστε τις οδηγίες χρήσης Siga las instrucciones de uso Noudata käyttöohjeita Respecter la notice d'utilisation Kövesse a használati utasítást Seguire le Istruzioni per l'uso 사용 설명서에 따를 것 Gebruiksaanwijzing volgen Postępować zgodnie z instrukcją użycia Seguir as Instruções de Utilização Urmați instrucțiunile de utilizare Соблюдайте инструкции по применению Följ bruksanvisningen Kullanna talimatlarını takip edin 遵守《使 用说明书》 遵守使用說明</p>

LANGUAGES INCLUDED IN THIS INSTRUCTIONS FOR USE

English	8
Deutsch	16
العربية	24
Čeština	32
Dansk	40
Ελληνικά	48
Español	56
Suomi	64
Français	72
Magyar	80
Italiano	88
한국어	96
Nederlands	104
Polski	112
Português	120
Română	128
Русский	136
Svenska	144
Türkçe	152
简体中文	160
繁體中文	168

EN

DE

AR

CS

DA

EL

ES

FI

FR

HU

IT

KO

NL

PL

PT

RO

RU

SV

TR

ZHS

ZHT

TABLE OF CONTENTS

1.	INTRODUCTION	9
1.1.	INTENDED PURPOSE	9
1.2.	INDICATIONS FOR USE	9
1.3.	TARGET USER GROUPS	9
1.4.	CONTRAINDICATIONS	9
1.5.	CONVENTIONS	10
1.6.	GENERAL WARNINGS	10
2.	DEVICE DESCRIPTION	11
2.1.	SETUP	12
2.2.	MOUNTING THE BACKUP PUMP DRIVE	12
2.3.	USING THE ADJUSTABLE CLAMPING HANDLE	13
2.4.	POSITIONING THE PUMP DRIVE	13
2.5.	REMOVING THE BACKUP PUMP DRIVE	13
3.	HANDLING	14
3.1.	CLEANING AND MAINTENANCE	14
3.2.	STORAGE AND DISPOSAL	14

1. INTRODUCTION

The purpose of these Instructions for Use is to instruct qualified medical staff in the function and maintenance of the Holder for Backup Pump Drive. All users of the Holder for Backup Pump Drive must have read these Instructions for Use prior to using this holder. The Holder for Backup Pump Drive is a class I device according to the Regulation (EU) 2017/745 on medical devices (MDR).

1.1. INTENDED PURPOSE

The Holder for Backup Pump Drive is a non-therapeutic part of the Xenios System (deltastream MDC console, iLA active console, i-cor console or Xenios console). It is intended to only be used as a passive IV pole mount for the console backup pump drive. The Holder for Backup Pump Drive is used to store the backup pump drive. The backup pump drive is put onto the holder and this can be mounted at a trolley pole in operating room (OR) or intensive care unit (ICU) environment.

1.2. INDICATIONS FOR USE

The Holder for Backup Pump Drive in combination with the trolley can be used to provide a standby backup pump drive during the treatment of neonates, pediatric and adult patients.

1.3. TARGET USER GROUPS

The Holder for Backup Pump Drive may only be used by qualified medical staff required to work with it.

1.4. CONTRAINDICATIONS

There are no known contraindications, complications or interactions.

Note: Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

1.5. CONVENTIONS

The following symbol appears in these Instructions for Use and on the Holder for Backup Pump Drive packaging and labelling.

CAUTION

WARNING indicates a hazardous situation which, if not avoided, could result in death or serious injury.

Note: Notes are advisory comments or recommendations regarding practices or procedures.

1.6. GENERAL WARNINGS

This section contains general warnings statements regarding the use and maintenance of the Holder for Backup Pump Drive. It is not a complete summary and additional warning statements specific to pertinent topics can be found within these Instructions for Use.

Note: Refer to the console and patient kit Instructions for Use for information regarding Warnings and Cautions.

CAUTION

Before attaching the backup pump drive to the Holder for Backup Pump Drive, make sure that the Holder for Backup Pump Drive is securely and firmly attached to the pole. Otherwise, the Holder for Backup Pump Drive may fall and will damage the backup pump drive or other components located below the Holder for Backup Pump Drive.

Ensure that the backup pump drive is completely seated on the Holder for Backup Pump Drive (pin with radial groove) and secured with the clamping screw. Otherwise, the backup pump drive may fall and damage components located below the backup pump drive.

Position the Holder for Backup Pump Drive with the backup pump drive in a way that the components used for treatment (oxygenator and pump drive/pump head) are not affected.

CAUTION

When adjusting the correct position of the Holder for Backup Pump Drive (movable element with backup pump drive), make sure that hand or fingers will not be pinched.

Ensure that the Holder for Backup Pump Drive and the individual components are mounted and fixed securely and screws are tightened. All screws shall be tightened and loosened by hand only.

Do not expose the Holder for Backup Pump Drive to high mechanical stress.

Do not use the Holder for Backup Pump Drive if it is damaged (e.g. damages on screws, and holder frame).

If the Holder for Backup Pump Drive is dropped or falls down, the Holder for Backup Pump Drive and all fastening components must be checked for damage. The damaged holder or components should not be used further and must be exchanged.

Note: The Holder for Backup Pump Drive may only be used to mount the backup pump drive while backup pump drive is not active.

In addition to following the Instructions for Use, users must observe the rules and regulations regarding occupational safety, accident prevention, and environmental protection.

2. DEVICE DESCRIPTION

The Holder for Backup Pump Drive is a holder for the mounting and positioning of a console backup pump drive.

Item	Weight	Item number	Max load	Material
Holder for Backup Pump Drive	approx. 680 g	38350411	5 kg	Aluminum

For SVHC information according to Article 33 of Regulation (EC) No.1907/2006 ("REACH") please visit this page: www.xenios-ag.com/SVHC.

2.1. SETUP

CAUTION

Before attaching the Holder for Backup Pump Drive, make sure the mounting pole is stable.

Attach the Holder for Backup Pump Drive to a pole (e.g. IV pole) in an orientation that ensures that enough space for the backup pump drive is available. Make certain that the clamping screw is tightened firmly so that the holder cannot be moved on the pole.

2.2. MOUNTING THE BACKUP PUMP DRIVE

After the Holder for Backup Pump Drive is securely attached to the pole, the backup pump drive can be mounted on the notched pin of the holder.

To do this, first loosen the securing screw on the bottom end of the backup pump drive handle. This allows the socket located at the bottom end of the handle to be slid completely over the pin of the holder. The securing screw is then tightened into the radial groove of the mounting pin. To remove the Holder for Backup Pump Drive, loosen the clamping screw.

Pos. in fig. 1	Description
1	Securing screw
2	Pin with radial groove

2.3. USING THE ADJUSTABLE CLAMPING HANDLE

Adjustable clamping handles can be tightened by two methods: either by rotating the handle clockwise until it no longer turns easily, or by using the semi-ratcheting function of the handle. The semi-ratcheting function is particularly useful when space is limited. The operating button is depressed, the lever lifted (up) and the handle is rotated as far as possible. Releasing the handle returns the lever to the neutral position, from which it can be pushed back to the start position for the next round of tightening via rotation with a lifted lever.

Pos. in fig. 2	Description
1	Operating button
2	Lever
3	Hinge

2.4. POSITIONING THE PUMP DRIVE

The positioning of the backup pump drive can be set along two axes. Either the hinge can be adjusted and secured using the adjustable clamping handle, or the backup pump drive can be rotated on the mounting pin and fixed in place using the securing screw. After the desired final arrangement of the backup pump drive and the holder have been achieved, the corresponding screws on the holder and backup pump drive must be securely tightened [figure 2].

2.5. REMOVING THE BACKUP PUMP DRIVE

Before the backup pump drive can be attached to the console (e.g. onto the compact holder), it must be removed from the Holder for Backup Pump Drive. To do this, loosen the securing screw on the backup pump drive handle and slide the backup pump drive off of the mounting pin.

3. HANDLING

This section provides important information about maintenance and storing the Holder for Backup Pump Drive.

3.1. CLEANING AND MAINTENANCE

To maintain reliable functionality and prevent infection, contamination, or malfunction, the Holder for Backup Pump Drive must be cleaned and checked for intactness before each use.

For disinfection, use a medical grade disinfectant, such as:

- Bacilol Plus™ from Bode Chemie GmbH
- Incidin Schaum™ from Ecolab Healthcare GmbH
- Meliseptol™ from B. Braun Melsungen AG

Further approved cleaning solutions can be found in the console Instructions for Use.

Maintenance of the Holder for Backup Pump Drive is recommended as part of the console's maintenance interval. If any problem occurs, contact the manufacturer.

3.2. STORAGE AND DISPOSAL

Store and transport the Holder for Backup Pump Drive in clean and dry condition.

Follow all environmental regulations regarding waste disposal.

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